

EZP Urinal Holder / Nebbo

Cleaning and Disinfection Instructions

For information on products for cleaning/disinfection go to [epa.gov/coronavirus](https://www.epa.gov/coronavirus)

The EZP Urinal Holder/Nebbo is a dedicated disposable device classified as a non-critical item that requires low-level disinfection.

Recommended Frequency of Cleaning:

1. When visibly soiled.
2. Before storage.

Recommended Cleaning and Disinfection Agents:

1. EPA registered hospital disinfectant and detergent cleaner.
2. Or a combination detergent/EPA registered hospital disinfectant.

Cleaning Instructions:

1. Remove all foreign material with water, mechanical action and detergent product.
2. Apply EPA registered disinfectant product to all surfaces of the EZP Urinal Holder.
3. Some facilities use a product that has detergent and disinfectant combined. In this case step 1 and 2 above will be combined.
4. Follow the manufacturers instruction of the disinfectant product you are using for the contact time (i.e. - wet time, dwell time).
5. Air dry.

Reference

APIC Text- Rutala WA, Weber DJ, Cleaning, Disinfection, and Sterilization. Infection Control and Applied Epidemiology, Inc. Fourth Edition 2014 June, 31: 2-6

June 6, 2021